



Clear Advantage^{Bite Splints}



CLEAR ADVANTAGE

CLEAR ADVANTAGE^{PLUS}

Through exhaustive research, DISTINCTIVE DENTAL STUDIO, LTD. has developed an appliance in order to prevent damage from para-functional habits and to protect existing dental restorations all while keeping the patients comfort a priority. As a result, THE DISTINCTIVE TEAM would like to introduce our new line of *Clear Advantage Bite Splints*. The *Clear Advantage* is fabricated entirely from a slightly flexible, thermosensitive material which allows custom adaptation in order to achieve optimal patient comfort. The *Clear Advantage^{Plus}* is dual processed with the highly dense injected Methylmethacrylate resin as the occlusal surface and the flexible thermosensitive resin as the liner. This combination allows for excellent occlusal wear resistance combined with the ultimate in patient comfort.

HIGH STRENGTH

PRACTICE^{BUILDER}

VIRTUALLY UNBREAKABLE

ACCURATE^{FIT}

REDUCED^{CHAIR TIME}

MAINTAINS TRANSLUCENCY

OPTIMAL^{COMFORT}

THE DISTINCTIVE ADVANTAGE

Clear Advantage^{Bite Splints}

PRODUCT DESCRIPTION:

Clear Advantage: A bite splint fabricated entirely from a slightly flexible, thermosensitive material which allows for custom adaptation to ensure patient comfort.

Clear Advantage^{Plus}: A bite splint that is dual processed with the highly dense injected Methymethacrylate resin as the occlusal surface and the flexible thermosensitive resin as the liner.

ADA INSURANCE CODE: 9940

INDICATIONS:

Clear Advantage: For patients with a light to moderate parafunctional habit or for protection of significantly restored dentition.

Clear Advantage^{Plus}: For patients with a highly destructive parafunctional habit due to the excellent occlusal wear resistance of the injected methymethacrylate resin.

CONTRA-INDICATIONS:

Clear Advantage: Not recommended for patients with a highly destructive parafunctional habit.

Clear Advantage^{Plus}: has no known contra-indications.

RECOMMENDED CLEANING: Conventional denture cleanser and/or toothpaste and toothbrush is recommended.

RECOMMENDED IMPRESSION TECHNIQUE/MATERIAL: Alginate is recommended if it can be poured immediately and separated no later than 1 hour after being poured. Vinyl Polysiloxane is the second choice. No putty wash materials should be used.

DOCTOR BENEFITS:

- Reduces Chair Time
- Improved Patient Compliance
- Patient Satisfaction
- Practice Builder

PATIENT BENEFITS:

- Optimum Comfort
- Protects Their Investment
- Long Lasting
- Ease of Insertion

HELPFUL HINT: For ease of insertion, place the appliance in a container of warm tap water to increase the flexibility of the thermo-plastic resin.



DISTINCTIVE DENTAL STUDIO, LTD.

1504 WALL ST. NAPERVILLE, IL 60563

PH: 800.552.7890 FX: 630.369.6539

WWW.DDSLTDLAB.COM

FOR FURTHER INFORMATION OR TO ARRANGE A PICKUP,
PLEASE CALL US AT 1.800.552.7890



DDS

Experience

THE DISTINCTIVE ADVANTAGE

HIGH STRENGTH
PRACTICE BUILDER

**PROTECTS EXISTING
DENTAL RESTORATIONS**

ACCURATE FIT

REDUCED CHAIR TIME

MAINTAINS TRANSLUCENCY

OPTIMAL COMFORT

DDS
A DISTINCTIVE
ORIGINAL

1.800.552.7890

DISTINCTIVE DENTAL STUDIO, LTD.

Clear Advantage Plus Bite Splints

Clear Advantage *Plus* Bite Splints

A bite splint that is dual processed with the highly dense injected Methlymethacrylate resin as the occlusal surface and the flexible thermosensitive resin as the liner.

Experience

“The Distinctive Advantage”

by calling us at 800-552-7890

for further information or to

arrange a pick-up!



DISTINCTIVE DENTAL STUDIO, LTD.

1504 WALL ST., NAPERVILLE, IL

Ph: 800.552.7890 Fx: 630.369.6539